

Element DCX


Barcode Scanner Cleaning




This document is intended for Antech and Biovet Canada employees and customers.

Element DCX


1. Turn the analyzer off, remove the slide holder and move the Transfer Bar to the right with your finger as shown below.

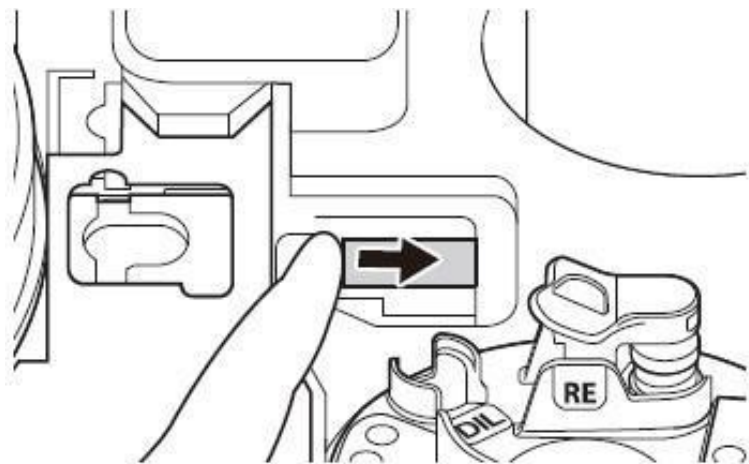

2. Down in the space below where the slides sit, blow some compressed air into the hole while pointing the straw to the right as shown below.

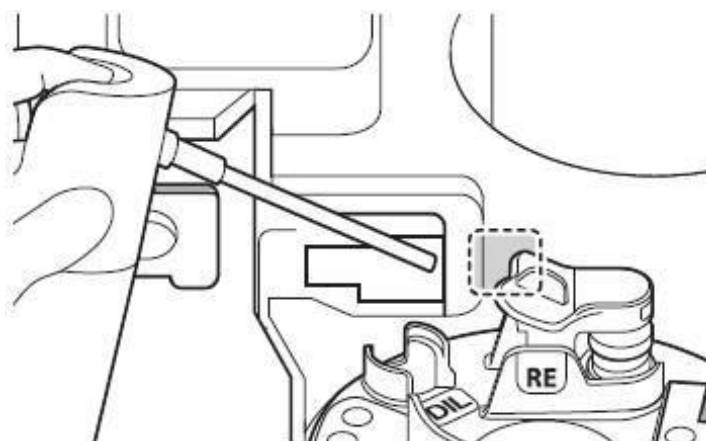

Element DCX

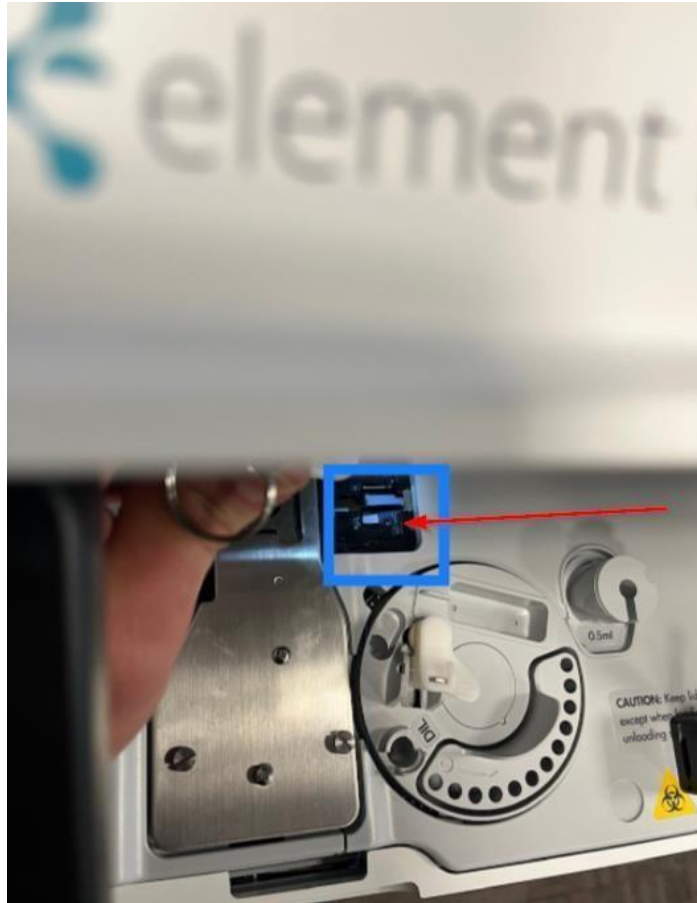

3. If the camera is very dirty you may need to press a dry cotton swab on to the tiny barcode scanner camera lens shown in the picture below and rotate it around to clean the lens. A flashlight is helpful in seeing the camera.